



# HELITACK STORK™

Made in USA  
of US and foreign  
components

## INSPECTION

Inspect the equipment according to your department's policy for inspecting life safety equipment. Inspect the equipment prior to entry into service, after each use, and at least once every 12 months. The equipment should be thoroughly inspected by an inspector that meets your department's training standard for inspection of life safety equipment. Keep a record of the date, person performing the inspection and results, as well as the date of first use, name of users and any other pertinent information necessary to keep accurate track of the equipment's usage history in the equipment log or on a tag that attaches to the equipment. Each user should be trained in equipment inspection and should inspect the equipment before each use.

When inspecting the equipment, check the webbing and rope for cuts, worn or frayed areas, broken fibers, soft or hard spots, discoloration, or melted fibers. Check the stitching for pulled threads, abrasion, or breaks. Check the hardware for damage, sharp edges, and improper operation. If any of the above is noted, or if the equipment has been subjected to shock loads, fall loads, or abuse other than normal use, remove the equipment from service and destroy it. If there is any doubt about the serviceability of the equipment, remove the equipment from service and destroy it.

The service life of equipment depends greatly on the type of use and the environment of use. Because these factors vary greatly, a precise service life of the equipment cannot be provided.

## USE

The CMC Helitack Stork is intended to be used as pediatric rescue adjunct during helicopter hoist or other technical rescue operations. The ends of the Stork can be opened to accommodate a pediatric backboard, otherwise the restraint system located on the underside of the bottom flap can be used to secure the patient. The top ventilation flap adjacent to the clear panel can be opened to provide ventilation as necessary.

## ⚠ WARNING

- SERIOUS INJURY OR DEATH MAY RESULT FROM THE IMPROPER USE OF THIS EQUIPMENT.
- THIS EQUIPMENT HAS BEEN DESIGNED AND MANUFACTURED FOR USE BY EXPERIENCED PROFESSIONALS ONLY.
- DO NOT ATTEMPT TO USE THIS EQUIPMENT WITHOUT PRIOR TRAINING.
- THOROUGHLY READ AND UNDERSTAND ALL LABELS AND INSTRUCTIONS BEFORE USE.
- USE, INSPECT AND REPAIR ONLY IN ACCORDANCE WITH MANUFACTURER'S INSTRUCTIONS.

## CMC Rescue, Inc.

6740 Cortona Drive, Goleta CA 93117  
USA  
805-562-9120 / 800-235-5741  
cmcpro.com

## ISO 9001 Certified

©2018 CMC Control No. 202200.02.012418

## CARRYING, MAINTENANCE & STORAGE

During use, carrying and storage keep the equipment away from acids, alkalis, exhaust emissions, rust and strong chemicals. Do not expose the equipment to flame or high temperatures. Carry the equipment where it will be protected as the equipment could melt or burn and fail if exposed to flame or high temperatures.

If the equipment becomes soiled, it can be washed in cold water with a mild detergent that is safe for use with nylon and polyester. Dry out of direct sunlight. Do not dry in an automatic dryer. Store in a cool, dry location. Do not store where the equipment may be exposed to moist air, particularly where dissimilar metals are stored together.


## REPAIR

All repair work shall be performed by the manufacturer. All other work or modifications void the warranty and releases CMC from all liability and responsibility as the manufacturer.

## SAMPLE INSPECTION AND MAINTENANCE LOG


The sample log suggests records that should be maintained by the purchaser or user of life safety equipment.

Equipment Inspection and Maintenance Log			
Item _____ # _____		Date in Service _____	
Brand/Model _____		Strength _____	
Date	How Used or Maintained	Comments	Name

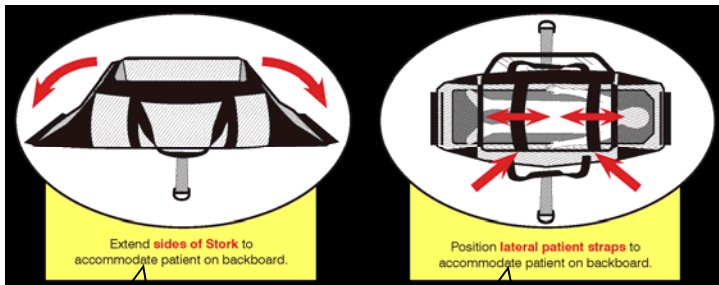

Extend sides of Stork to accommodate patient on backboard.

Position lateral patient straps to accommodate patient on backboard.

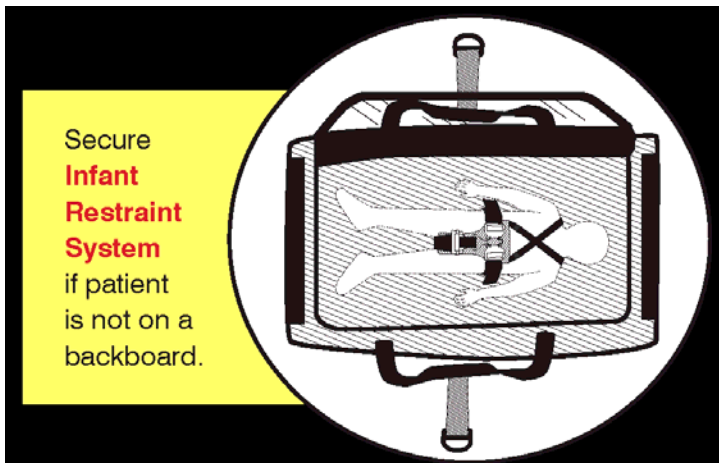

Secure Infant Restraint System if patient is not on a backboard.